Description • Descripción

Porous Regenerative Membrane - Membracel consists of interwoven crystalline cellulose microfibrils, has artificially developed pores, fine texture and uniform structure. These features allow protection from skin lesions and promote tissue regeneration by acceleration of granulation tissue formation and subsequent epithelialization.

Product expected action: Pain relief, injury protection, drainage of secretions, reduction of exposed regions to infectious agents and acceleration of the healing process.

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Indications • Indicaciones

Porous Regenerative Membrane - Membracel acts as a temporary replacement of the skin and is suitable for injury treatments resulting from the loss of epithelium that are characterized as superficial or deep wounds, with little or abundant exudation, such as second-degree burns, dermal abrasions, excoriation, recipient and donor areas for skin grafts, nail beds (post-nail excision), pressure sores, varicose ulcers of stasis, arterial ulcers, plantar perforating maladies, epidermolysis bullosa, post-physical cauterization (cryotherapy, thermal cauterization, CO2 laser and Erbium).

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Contraindications • Contraindicaciones

Membracel should not be used to control bleeding, malignant lesions or suspected malignancy.

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Choosing the membrane •
Escogiendo la membrana

Choosing the right **Porous Regenerative Membrane - Membracel** is essential for treatment.

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Size • Dimensión

Consider the size of the wound to choose the most suitable size, so that the Membracel cover the entire exposed injury area and exceeding by at least 1/3 in. (1 cm) its edges.

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Consider the type of wound, the volume and the characteristics of the existing exudate in the wound to choose the ideal pore size:

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MEDIUM – 1 to 2 mm
Suitable for low and/or moderate exudation. Used in most cases.

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LARGE – 2 to 3 mm
Indicated in cases of abundant and/or thick exudation.

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The artificially produced pores in **Porous Regenerative Membrane - Membracel** allow free passage of the exudate from the lesion bed to the secondary dressing, which must have absorbent characteristics.

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Method of use • Modo de usar

- Clean the lesion with saline solution (SSI 0.9%) in a jet or other specific solution. If necessary, perform the debridement for removal of non-viable tissues.
- Remove **Porous Regenerative Membrane - Membracel** from the envelope and place it over the lesion, in order to cover the lesion with an excess of 1/3 in. (1 cm) beyond its edges. The product can be applied using either side;
- With the use of sterile tweezers and sterile gauze, perfectly settle the product over the injured area, gently pressing gauze on the membrane.
- If necessary, apply an absorbent secondary dressing on **Porous Regenerative Membrane - Membracel**, such as sterile gauze. For fixation, use bands, compression bandages and microporous tape.

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For greater treatment efficiency, it is recommended to change the secondary dressing whenever saturated with exudate or has lost absorption capacity, or according with instructions as provided by your health care professional.

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Membracel Frequency of Change • Intervalos de cambio de la membrana

1 The change frequency will depend on the condition of the wound and the volume of exudate. Initially it is normal to change/ replace **Porous Regenerative Membrane - Membracel** every 5 days. Over time, this interval will naturally increase, due to the reduction in the exudate volume secreted by the injury.

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2 The **Porous Regenerative Membrane - Membracel** can be maintained for up to 12 days if it does not occur peel off, ruptures, signs of infection, acute pain, foul odor or stench.

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1 To remove secondary dressing applied over the membrane.

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2 Cleaning the wound area without removing the membrane using saline solution (0.9%) and a gauze to compress and remove the excess exudate under the membrane, through its pores. In this procedure, avoid too moisten the membrane to prevent its detachment.

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3 Place gauze or other sterile absorbent secondary dressing. Secure the secondary dressing with microporous adhesive tape or band, so that it is lightly pressed over the membrane/lesion.

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If you need to change the **Porous Regenerative Membrane - Membracel**, adopt the following:

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1 Moisten the adhered part of the membrane with gauze soaked in saline solution (0.9%) for about 4 to 5 minutes.

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2 Detach Membracel's edges, gently rubbing the gauze soaked in saline solution (0.9%) in the periphery-center direction of the lesion until Membracel is completely removed;

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3 Clean the wound bed using saline solution (0.9%) in jet or another specific solution. Apply a new membrane, observing the procedures described in Method of use.

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Special care • Cuidados especiales

1 Never use oily products, ointments or creams with **Porous Regenerative Membrane - Membracel**, since this practice prevents its attachment to the lesion. Should further topical treatment be necessary, products in the form of Solutions are most recommended. However, avoid use of hydrogen peroxide as this will cause chemical degradation of **Porous Regenerative Membrane - Membracel**.

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2 Avoid wetting the membrane during bath (protect the dressing with waterproof material).

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3 Caution: sterilization is guaranteed as long as the packaging is not damaged or opened before use. Do not use **Porous Regenerative Membrane - Membracel** if the packaging is broken and / or date has expired.

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4 Systemic antimicrobial therapy may be prescribed by your physician when the wound presents infection. In this case, **Porous Regenerative Membrane - Membracel** can be used in conjunction with systemic antibiotic.

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5 If it is necessary to cut **Porous Regenerative Membrane - Membracel**, it is recommended to use sterile scissors.

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6 The use of **Porous Regenerative Membrane - Membracel** should be supervised by a healthcare professional.

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Storage • Almacenamiento

This product, when stored at temperatures below 25° Celsius, away from light and moisture, is good for 60 months from date of manufacture.

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Packs with 1 or 10 **Porous Regenerative Membranes**.

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Lot number, manufacturing date and expiration date:
see packing.

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